



Shower Chair



VB540B

Usage and Maintenance Instructions

This file is available to view and download as a PDF at www.aidapt.co.uk. Sight impaired customers can use a free PDF Reader (such as adobe.com/reader) to zoom in and increase the text size for improved readability.

INTRODUCTION

Thank you for deciding to purchase the Shower Chair from Aidapt. When used correctly it is designed to give many years of reliable trouble-free service.

NB. This equipment must be installed by a competent person.

120 kg (18 3/4 stone) weight limit.

Do not exceed the weight limit stated – doing so could put the user at risk.

BEFORE USE

- Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the product.
- Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier immediately for support.

PARTS LIST

Seat Board x 1

Vaulted Washer (Ø6) x 5

Screw (M6 x 35mm) x 5

Leg Assembly A x 1

Leg Assembly B x 1

ASSEMBLY

1. Lay the seat board flat, making sure that the arrow is pointing forward (see fig. 1)

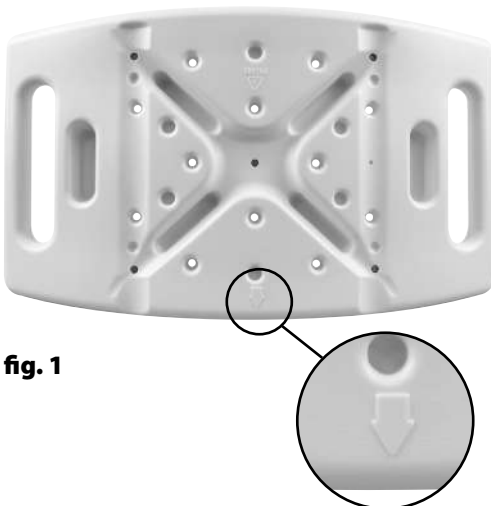


fig. 1

2. Identify leg assembly A. (When stood up Leg Assembly A has a flat top and a cut-out on the underside of the tube) (see fig. 2)

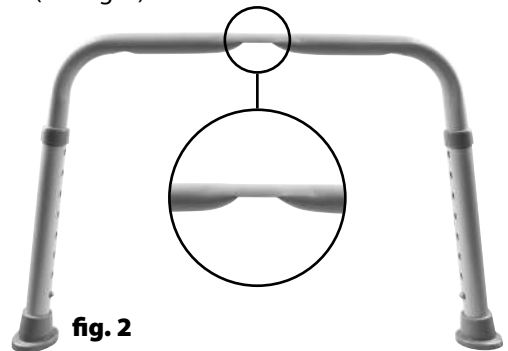


fig. 2

ASSEMBLY (CONTINUED)

3. Attach leg assembly A to the seat board, using 2 x M6 screws and 2 x vaulted washers (see fig. 3)

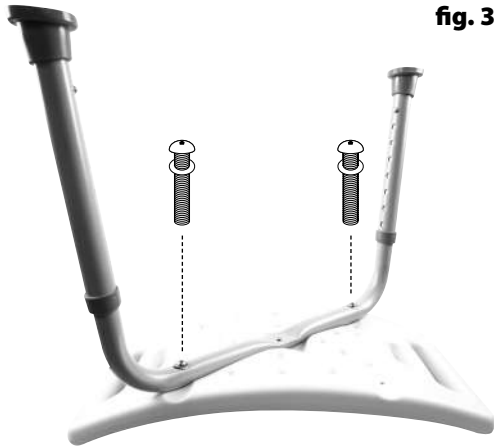


fig. 3

4. Identify leg assembly B. (When stood up Leg Assembly B has a cut-out on the top of the tube and the underside of the tube is flat (see fig. 4)

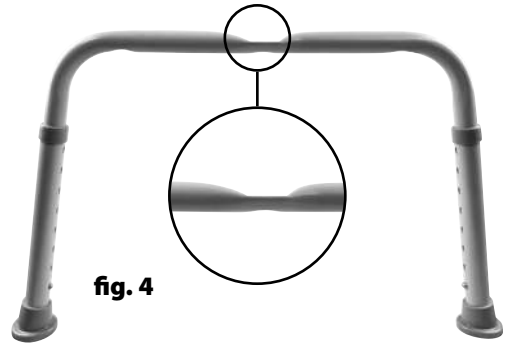


fig. 4

5. Attach leg assembly B to the seat board, using 2 x M6 screws and 2 x vaulted washers (see fig. 5)

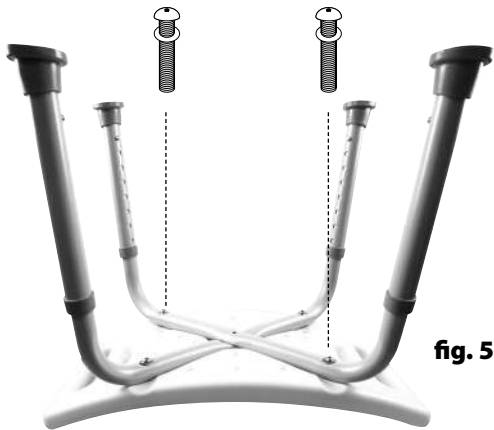


fig. 5

6. Secure both leg assemblies to the seat board using 1 x M6 screw and 1 x vaulted washer (see fig. 6)

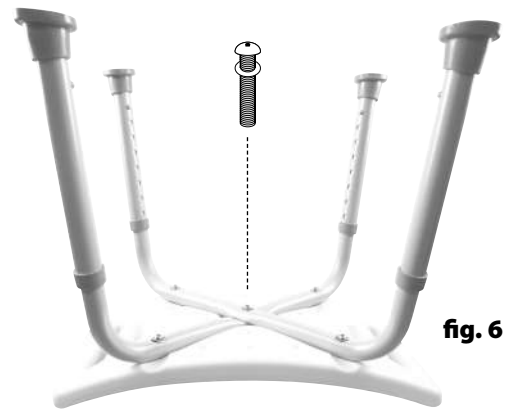


fig. 6

NB. We do not recommend the use of a battery operated screw driver when assembling this product, but make sure screws are tight and the seat is stable.

HEIGHT ADJUSTMENT

1. Depress the securing pin and twist the extension leg to the side to allow you to slide the extension leg up or down to the desired height.
2. Rotate the leg back to allow the securing pin to pop out of the hole and fix the height at the correct setting.
3. Repeat steps 1 and 2 above for each leg, **ensuring that all four legs are set to the same height setting.**

CLEANING

All of the product components are manufactured from quality materials. However, it is very important that you only clean your shower chairs and stools using a non-abrasive cleaner or mild detergent with a soft cloth, avoid using lime scale remover. Abrasive cleaners e.g. AJAX and or abrasive cleaning pads will seriously damage your product beyond repair. Do not exceed 50°C when cleaning the seat as this will permanently damage the plastic.

N.B. Regular daily cleaning will reduce the buildup of limescale on your product.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety. This includes:

- Bent or damaged tubes or cracks at the tube joints.
- Elongated adjustment holes.
- Loose or worn fasteners
- Smooth, worn or split ferrules (rubber tips).

Replace all worn, missing or damaged parts where necessary. If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CARE & MAINTENANCE

Please make a check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX

Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk

Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk